



Instructions for AttenuRad CT Eye Shield and CT Thyroid Shield

Step One

It is important to run the scout topogram prior to placing shield.

If the scanner detects increased density from the presence of the shield, it will automatically increase power to compensate as if the patient is larger. This reduces the effectiveness of the shield and will result in lower dose savings.



Step Two

Eye (ARE)

With patient in position for scan, be sure patient has closed eyes. Place yellow AttenuRad Eye Shield (ARE) as shown using adhesive provided on the bridge of the nose.

Thyroid (ART)

With patient in position for scan, place yellow AttenuRad Thyroid Shield (ART) as shown using adhesive provided.

PH: (724) 845-7028

FAX: (724) 845-5439

**1129 Industrial Park Road
Vandergrift, PA 15690**

www.fandlmedicalproducts.com